FitScale W1 BT

EN – INSTRUCTIONS FOR USE


TrueLife FitScale W1 BT wireless scales measure body weight by bioelectrical impedance analysis (BIA). This technology combined with BIA is used to measure body fat and composition in order to obtain its data. The scales are for home use only.

- In order to achieve consistent and accurate results, the scales should be used at the same time each day and, if possible, under approximately the same conditions. Avoid use under the following circumstances:

1. During or immediately after a physical workout
2. Immediately after a shower, bath or sauna
3. After the consumption of alcohol
4. During or immediately after eating
5. When affected by a fever


- The unit of measurement can be altered from kilograms (kg) to pounds (lb.) in the mobile application. This will also alter measurements of height and distance (meters/feet).

- Ensure the scales are placed on a flat surface or on a solid mat. Reset the scales before their use. They should initially indicate 0.0 kg.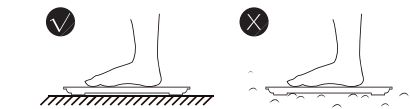


Profile of scales on level ground and on uneven ground

- To measure body fat and other parameters correctly: Stand upright on the scales with both legs parallel. They will automatically switch on when the surface is pressed. Stand upright and barefooted. The weight of socks will ensure inaccurate results. Even dirty feet can affect accurate measurement.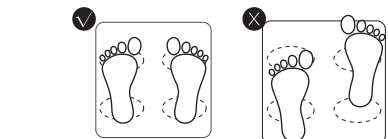

Graphical representation of the feet on the weighing area


Possible complications: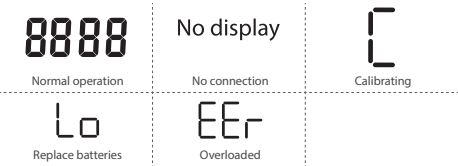


Installing the application and measuring:

To measure all body proportions, such as subcutaneous fat, connect FitScale to your mobile phone or tablet. This can be achieved by connecting TrueLife FitScale W1 BT to your mobile phone via Bluetooth. The easiest way to obtain the mobile app is to download it from Appstore/Google Play or by scanning the QR code on the box or manual. Follow the steps below to install:

1. Downloading the application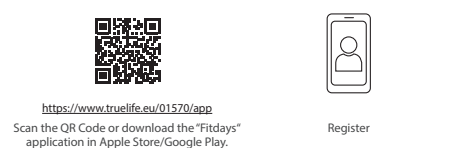

2. Pair the scales with the application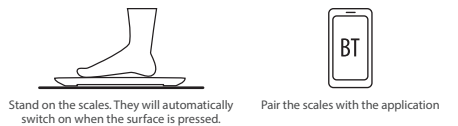

Description of the data on the display:


Weighing process

Do not step off the scale while the weighing is in progress. Wait until the weighing process is complete. The display will show the weighing results, which include body weight, body fat ratio and BMI. Each result will be displayed for approxima-

tely 3 seconds.

Weighing without using the app will display the measured values on the LCD screen as following: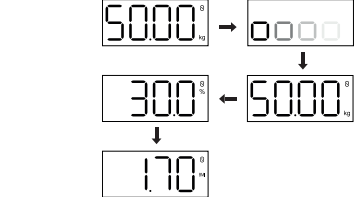

Package contents:

The contents of your TrueLife FitScale W1 BT scales are:

1. TrueLife FitScale W1 BT
2. 2x 1.5V AAA batteries
3. User Manual + Safety Regulations

Basic specifications:

Dimensions: 26 cm x 26 cm x 2.4 cm
Weight: 1.26 kg
Batteries: 2x 1.5V AAA
Weight range: up to 180 kg
Weight deviation: 0.1 kg
Units: kg/lb

Manufacturer's responsibility:

TrueLife products are under warranty for manufacturing defects. Elem6 s.r.o. are, in the event of such defects, under obligation to repair or replace the defective part or product with a comparable part or product at its own discretion. Other alterations, conversions or further unauthorized manipulation of the product will result in the termination of the warranty. This shall also occur should continued use of defected parts, negligence or other errors in the product usage result in loss or damage. The warranty does not apply to damage resulting from standard usage, accidental or normal wear and tear. Elem6 s.r.o. assumes no responsibility for any accident, injury, death, loss or other claims relating to or arising out of the use of this product. Elem6 s.r.o. shall under no circumstance be made liable for any incidental or consequential damages resulting from or during the use of this product or any of its components. Repayment or replacement of spare parts and/or products may be subject to charges for transportation, handling, replacement and/or replenishment.

EU Declaration of Conformity

elem6 s.r.o. hereby declares that the TrueLife FitScaleW1 BT complies with the essential requirements and other relevant provisions of Directive 2014/53/EU and 2014/30/EU. All TrueLife products are intended for sale without restriction in Germany, the Czech Republic, Slovakia, Poland, Hungary and other EU member states. The full Declaration of Conformity can be downloaded from www.truelife.eu/support/doc
Frequency bands in which the radio equipment operates: 2.4 ~ 2.48 GHz

SAFETY PRECAUTIONS WHILST USING THE EQUIPMENT:

1. Carefully read the enclosed safety instructions.
2. For the correct handling of the product, read the latest version of the manual available at www.truelife.eu.
3. Print errors are reserved.

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