
truelife

Pulse

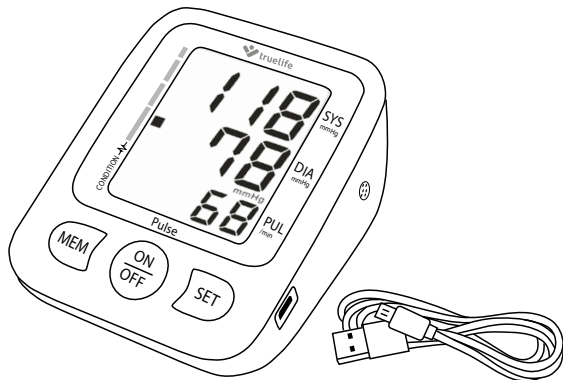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# EN – INSTRUCTIONS FOR USE

Before first use, the consumer must first read the instructions and safety precautions of the product.


## BOX CONTENTS

1× Blood pressure monitor  
1× Arm cuff  
4× AA 1.5V alkaline battery

1× Manual  
1× Safety Instructions


## 1. PRODUCT LAYOUT

- a. Cuff tube inlet
- b. Memory button "MEM"
- c. On / Off button
- d. Settings button "SET"
- e. USB power input
- f. Display
- g. USB cable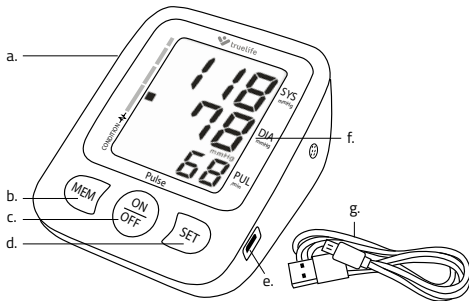


## 1.1 Display description

1. Systolic (upper) blood pressure
2. Diastolic (lower) blood pressure
3. Heart rate
4. WHO blood pressure classification
5. Battery
6. Motion during measurement
7. Irregular heartbeat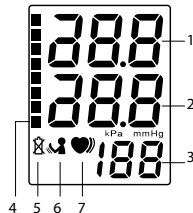

## 2. BATTERY INSTALLATION

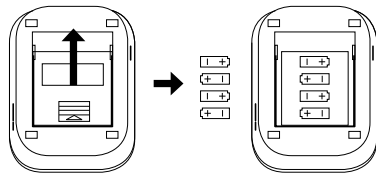Remove the battery cover on the back of the blood pressure monitor and insert 4 AA 1.5V alkaline batteries in the direction as shown by the arrow. Make sure the batteries are inserted in the correct direction of polarity as shown in the battery compartment. Do not use batteries after the expiry date.

The batteries must be replaced if the battery symbol appears on the display when the pressure monitor is turned on. The symbol indicates low battery power. At critically low battery power levels, the device will not take readings. The device is not suitable for use with rechargeable batteries.

If you do not intend to use the device for an extended period of time, remove the batteries.

### 3. USING AN AC ADAPTER

(The AC adapter is not included.)

Do not connect the adapter while the device is turned on.

When connected to an adapter, the device does not draw any battery power. If you disconnect the adapter from the device whilst taking a reading, you must plug the adapter back in again and repeat the reading.

#### a) Connecting

1. Insert the adapter connector into the USB jack on the right side of the monitor unit.
2. Plug the adapter into a 100-240 V socket.
3. The monitor will turn on when the ON/OFF button is pressed.

#### b) Disconnecting

1. Unplug the adapter.
2. Pull the adapter connector out of the USB jack on the side of the monitor.

**Note:** Only use the original adapter.

**Note:** When using the adapter, you can leave the batteries inside the device.

### 4. SETTINGS

To enter settings in off mode, hold down the SET button for 3 seconds. To switch between blood pressure indicators and stored readings, press the SET button. The set content flashes on the screen.

Press the MEM button to set the blood pressure units to mmHg or kPa. Press the ON / OFF button to save the setting.

If you want to clear the readings, press the MEM button and switch to 0. Pressing the ON / OFF button deletes all readings together.

**Note:** All readings on the monitor are deleted at once. If you choose to delete the measured values, we recommend that you take note of them somewhere else in case you need them in the future.

**Note:** Removing the battery will not delete the entries.

### 5. MEASURING BLOOD PRESSURE


Several factors play a role in measuring blood pressure. For the most accurate values, you must follow the following principles.

- Before taking a reading, rest for 5–10 minutes, avoid eating, drinking, smoking, exercising and bathing.
- If you feel a strong urge to urinate, do not take a reading but empty your bladder first. A full bladder can cause an increase in blood pressure of up to 10 mmHg.
- Connect the cuff hose to the body of the blood pressure monitor.
- Take off your outerwear. Keep on only thin clothing or remove all clothing from your arm.
- Put the sleeve of the cuff on your arm to a level above your elbow. Always measure on the same arm (ideally the left one).
- As the pressure changes during the day (usually the highest in the morning), take measurements at the same time each day.
- Repeated measurements accumulate blood in the arm and may cause measurement errors. The interval between measurements should be at least 1 minute. You can shorten the interval by raising your arm so that the blood drains.

#### 5.1 Attaching the cuff

Any effort to support your arm whilst taking a reading can increase pressure and distort measurement results. Hold the arm on which you are taking a reading so that the cuff is roughly at heart level. This will take the most accurate measurement.

1. Put the cuff onto your left arm so that the hose faces downwards and the lower part of the cuff so that it is approximately 1-2 cm above the elbow.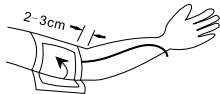


2. Make sure that the cuff is not tightened too tightly. You should be able to tuck two fingers between the cuff and arm.
3. Place your hand on the table with your palms pointed upwards so that you feel comfortable with the cuff roughly at heart level. Do not bend the hose. If it is not possible to use your left arm to take a reading, you

can also take measurements on your right arm. However, the reading should always be taken on the same arm.

## 6. TAKING A READING

- Press the ON/OFF button. All the symbols appear on the display. Then the display shows 0 and the device starts inflating the cuff. You can follow the cuff pressure on the display.
- Once the required cuff pressure has been reached, the pump stops inflating the cuff and the pressure gradually decreases.
- When the device detects a signal, a flashing heart symbol appears on the display. The symbol flashes once each time the heart beats.
- When the measurement is complete, a beep sounds and the display shows systolic and diastolic pressure and heart rate.
- The measurement results will remain on the display until the device is turned off. If idle, the device will turn itself off within 3 minutes. To display the results again, press the MEM button. Use the MEM and SET buttons to switch between the saved readings. The device can remember up to 60 readings. As soon as the device exceeds the memory capacity, the oldest readings will be overwritten. If the monitor does not measure the results correctly, the display will show the "E" icon.
- If for any reason it is necessary to interrupt the measurement (eg due to nausea), press the ON/OFF button. The device immediately relieves the cuff pressure automatically.

## 7. TABLE OF VALUES

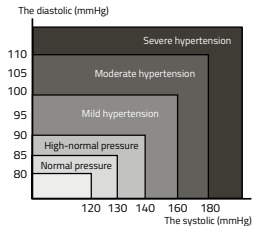Blood pressure levels	SBP mmHg	DBP mmHg
Optimal	< 120	< 80
Normal	120–129	80–84
High-normal	130–139	85–89
Grade 1 hypertension	140–159	90–99
Grade 3 hypertension	160–179	100–109
Grade 3 hypertension	>180	> 110

## 8. ERROR INDICATOR

### a) Symbol E-1

You may have an incorrectly attached cuff – Adjust it to the correct position and repeat the measurement

### b) Symbol E-4

Cuff hose is improperly inserted – Check tubing connection and adjust or reconnect if necessary. Then repeat the measurement.

Monitor or arm have moved whilst taking a reading – Keep your body and the monitor motionless whilst taking a reading. Repeat measurement.

The systolic pressure less than 30 mmHg – Manually increase the pressure for the desired result. Repeat the measurement.

### c) The E-5

Pressure is greater than 300 mmHg – The device is releasing air itself.

### d) Low Battery Symbol

Replace discharged batteries with new ones and repeat reading.

### e) Arm symbol in motion

The symbol appears when the hand is moving excessively.

## f) Heart Symbol

The symbol appears when the heart beats irregularly.

Measurement Method	Oscillometric
Pressure Measurement Range	0-300 mmHg
Heart Rate Measurement Range	40-180 bpm
Pressure Measurement Accuracy	$\pm 3$ mmHg
Heart Rate Measurement Accuracy	$\pm 5\%$
Memory	60 readings
Power Supply	4x AA Battery DC 6V or AC Adapter
AutoOff	After 3 Minutes
Pressure Monitor Weight	300 g (without cuff and batteries)
Pressure Monitor Dimensions	120 x 93 x 59 mm (without cuff)
Lifetime	approx. 10,000 readings
Operating Temperature	5-40 °C
Operating Humidity	10-90% RH

## 9. MAINTENANCE

- After use, store the blood pressure monitor in its packaging and protect it from damage.
- Store the blood pressure monitor in a dry environment.
- Keep the blood pressure monitor clean. Clean it with a dry cloth. Do not use abrasive or corrosive cleaners.
- Neither the blood pressure monitor nor any of the components should come into contact with water.
- Make sure the blood pressure monitor is off before cleaning. For cleaning it is possible to use distilled water with a 10 % bleach solution.
- Use a spray, moisten a soft cloth with bleach or detergent until it is soaked.
- Squeeze excess moisture from the fabric to prevent the unit from dripping or cuff getting wet.
- Wipe the entire surface of the cuff thoroughly. Make sure you clean the

inside and outside of the cuff properly. Be careful not to let any moisture get into the blood pressure monitor.

- Thoroughly wipe all parts of the cuff with a dry cloth. Unfold the cuff and allow it to air dry.
- Do not clean the blood pressure monitor or the cuff with diesel, thinner or petrol.
- Do not expose the device to extreme heat or cold, moisture, and direct sunlight.
- If you will not be using the device for more than 3 months, remove the batteries.
- Elem6 s.r.o. cannot be held responsible for any damage caused by insufficient maintenance and non-observance of the instructions in the manual.

## 10. PRECAUTIONS

- Use only the original charger supplied as an optional accessory to the blood pressure monitor.
- Do not use the charger in a high humidity environment. Never touch the charger with wet hands or while standing in water.
- When supplying power, leave enough space around the charger for air circulation. Do not cover the charger with paper or other objects that could lead to poor cooling. Do not use the charger stored in the transport packaging.
- Do not use the charger or device if it is obviously damaged. If the device is damaged, do not repair it yourself!
- In case of excessive heating, switch off the appliance immediately and disconnect it from the power supply.
- Charge the device only under supervision.

**Note:** This product is intended for use in homes and similar environments. We declare that if the TrueLife Pulse is used as a professional medical device, a safety inspection (BTK) must be carried out regularly at the following intervals: 24 months after purchase of the product, then every 12 months.

**IMPORTER:**

elem6 s.r.o., Braškovská 15, 16100 Prague 6,  
Czech Republic, Made in P.R.C.

C E 1639  **RoHS**

REF: AS-35A

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands  
Xiamen Ants-Bro Technology Co., Ltd, 4F, 5th Building,  
Technology business establishing center, No. 289 Wengjiao  
Road, Haicang District, Xiamen City, Fujian Province, China

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